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Intensiv HygienicTray

Product description

- Easily dismantled in 3 parts
- Suitable for the standard sterilisation tray
- Ideal arrangement of the 13 FG and 5 RA perforations for easy access to the instruments
- Stainless steel
- Dimensions
H x L x W: 32 x 50 x 30mm

Indications

- Intensiv HygienicTray may be used as an organizer for a specific indication treatment, i.e. prep bur block. In addition, it enables an easier handling of the instruments during the treatment and provides a support during the sterilisation process.

Practical recommendations

- To avoid any cross contamination, use disinfected tweezers to take the instruments.
- Use the completely foldable cover of Intensiv HygienicTray as a base during the treatment.

Maintenance and sterilisation

- For easy cleaning, dismantle the Intensiv HygienicTray completely (cover, instrument tray, base).
- The Intensiv HygienicTray is supplied nonsterile. Prior to first use and immediately after each use, the Intensiv HygienicTray must be disinfected, cleaned and sterilized.
- During the sterilisation keep the Intensiv HygienicTray separate from non-stainless steel instruments, such as rubber polishers and abrasive stones.
- If disinfection by immersion is being used, it is important to use only cleaning/disinfection solutions that provide corrosion protection and strictly observe the concentrations recommended by the manufacturer. Sterilization must be performed with validated procedures. If possible, use a single-pulsed or fractionated vacuum autoclave and subvacuum drying. Chemiclave sterilizers may also be used.
- Concerning the sterilization process we refer to the ISO standard 17664. We suggest to follow the indications below:
 - Cycles at 134°C:
T_{min} = 134°C – T_{max} = 138°C
Pressure = 3.15 bar abs
Time = 4 min (raisable)
 - Cycles at 121°C:
T_{min} = 121°C – T_{max} = 125°C
Pressure = 2.10 bar abs
Time = 16 min (raisable)

Intensiv HygienicTray
Ref. HT100

Additional models
Intensiv HygienicTray
Ref. HT200
Ref. HT300
Ref. HT3000

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